

*THE PALOMAR
PULSED LIGHT SYSTEMS
FOR REJUVELUX™ TREATMENTS*

*The RejuveLux process
uses pulsed light to treat
pigmented and vascular
lesions, for overall
improvement of skin tone
and texture.*



PALOMAR PULSED LIGHT SYSTEMS

PIGMENTED LESIONS INCLUDE:

- *Solar Lentigines:
More commonly known
as Sunspots, Age-spots or
Liver Spots*
- *Actinic Bronzing:
Overall browning of the skin.*



Solar Lentigines

VASCULAR LESIONS INCLUDE:

— *Telangiectasia*

Tiny capillaries that appear near the surface of the face. They may be red, blue or purple in color. Also known as Spider Veins.



Spider Veins

— *Rosacea*

Inflammation and swelling of the face.

PULSED LIGHT: HOW DOES IT WORK?

Pulses of concentrated light are directed onto the skin and absorbed by the melanin (for pigmented lesion treatment) and the blood (for vascular lesion treatment). Pulsed light systems are similar to lasers, but with a larger, more diffuse beam for safe, fast, and comfortable treatments.



PALOMAR PULSED LIGHT SYSTEMS

REJUVELUX PIGMENTED LESION TREATMENTS

The RejuveLux light pulse is directly absorbed by the pigment in the lesion. The light heats the pigment and breaks it up. The lesion darkens and crusts after treatment and is naturally shed from the skin after approximately ten days.



PALOMAR PULSED LIGHT SYSTEMS

REJUVELUX VASCULAR LESION TREATMENTS

RejuveLux light pulses heat the small capillaries that cause telangiectasia, Rosacea and other vascular lesions. The heat shrinks these capillaries, which may redden or become gray after treatment. They then fade from view after approximately ten days



PALOMAR PULSED LIGHT SYSTEMS

FREQUENTLY ASKED QUESTIONS

Will RejuveLux Treatments Hurt?

RejuveLux uses a unique long pulsewidth that delivers energy to the skin over a relatively long period of time without overheating the epidermis. This makes the pulses safer and more comfortable. You may experience a feeling similar to that of a rubber band being snapped on the skin, but most people do not report this sensation as painful

Are There Any Adverse Reactions to Treatment?

You may experience temporary redness and a mild “sunburn”- like effect that may last a few hours to a few days.

PALOMAR PULSED LIGHT SYSTEMS

*SEE THE RESULTS FOR YOURSELF:
REJUVELUX BEFORE AND AFTER PHOTOS*



Pre-treatment



After treatment

SOLAR LENTIGO

*Photos courtesy of:
David B. Vasily, M.D. Medical Director,
Aesthetica Cosmetic & Laser Center,
Bethlehem, PA*

PALOMAR PULSED LIGHT SYSTEMS

*SEE THE RESULTS FOR YOURSELF:
REJUVELUX BEFORE AND AFTER PHOTOS*



Pre-treatment



After treatment

SPIDER VEINS

*Photos courtesy of:
David B. Vasily, M.D. Medical Director,
Aesthetica Cosmetic & Laser Center,
Bethlehem, PA*

PALOMAR PULSED LIGHT SYSTEMS

*SEE THE RESULTS FOR YOURSELF:
REJUVELUX BEFORE AND AFTER PHOTOS*



Pre-treatment



After treatment

ROSACEA

*Photos courtesy of:
Photos courtesy of Randall Coverman, M.D.,
Altamonte Springs, FL*

PALOMAR PULSED LIGHT SYSTEMS

*SEE THE RESULTS FOR YOURSELF:
REJUVELUX BEFORE AND AFTER PHOTOS*



Pre-treatment



After treatment

ACTINIC BRONZING

*Photos courtesy of:
John Bruno, M.D.,
Ft. Myers FL*

PALOMAR PULSED LIGHT SYSTEMS

Ask your treatment provider about RejuveLux™ treatments with Palomar Pulsed Light Systems.

Palomar
MEDI LUX™
SYSTEM

Palomar
ESTE LUX™
SYSTEM

The best family of pulsed light systems in the world.

